



Communications &  
Power Industries  
Canada Inc.  
Satcom & Medical  
Products

45 River Drive  
Georgetown, Ontario, Canada L7G 2J4  
t +1 905 877 0161  
f +1 905 877 5327

## DAP Compatibility

The DAP chamber, when fitted with the proper interconnect cable, plugs directly into the Generator control board in the Generator. When ordering the DAP chamber from the DAP manufacturer, specify the CPI compatible interconnect cable, if available. This is a special cable terminated with a 9-pin male “D” connector that is designed to plug directly into the DAP connector on the Generator. If this cable is not available from the DAP device manufacturer, consult CPI product support for the required cable to connect the DAP chamber to the generator. Refer to the table below for the CPI cable assembly part numbers.

### **CMP 200® Series Generator**

The CMP 200® X-ray generator, when equipped with the DAP option, is compatible with the DAP devices listed in the table below.

<b>DAP DEVICE</b>	<b>INTERCONNECT CABLE</b>
PTW: PX-T11020	736145-00
Gammex-RMI: 841S	736146-00
VacuTec: VacuDAP 2004	Contact VacuTec for this cable assembly
Scanditronix (IBA): 120-131, 120-131HS	736148-00

### **Indico IQ® Series Generator**

The Indico IQ® X-ray generator, when equipped with the DAP option, is compatible with the DAP devices listed in the table below.

<b>DAP DEVICE</b>	<b>INTERCONNECT CABLE</b>
PTW: PX-T11020	736145-00
VacuTec: VacuDAP 2004	Contact VacuTec for this cable assembly
VacuTec: VacuDAP HS (High Resolution)	Contact VacuTec for this cable assembly

### **Indico 100® Series Generator**

The Indico 100® X-ray generator, when equipped with the DAP option, is compatible with the DAP devices listed in the table below.

<b>DAP DEVICE</b>	<b>INTERCONNECT CABLE</b>
PTW: PX-T11020	736145-00
Gammex-RMI: 841S	736146-00
VacuTec: VacuDAP 2004	Contact VacuTec for this cable assembly
Scanditronix -Wellhoefer: 120-131	736148-00